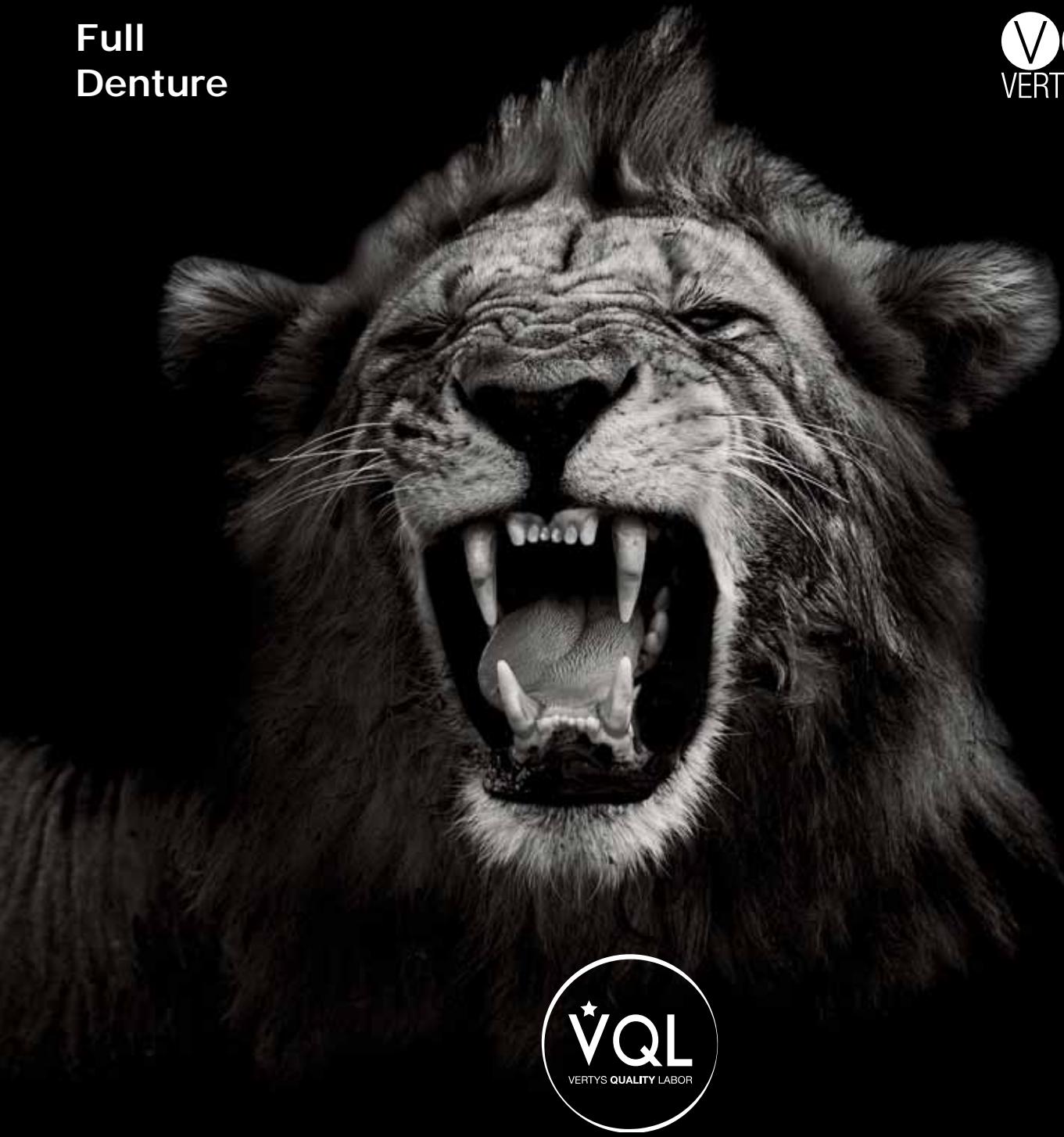
Full  
Denture

VXS  
VERTYSYSTEM®

W~~O~~RK DIFFERENT

Full  
Denture

IT EN ES

# OUR MISSION

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Vertysystem Group

*The task of a company today, it's not about just creating new equipment and consumable products more and more efficient, but especially providing some certain and tested work protocols to customers.*

*A Certified Protocol means having certain, repeatable passages and results, and a greater costs awareness. A.gree has been doing it all along thanks to its VERTYS QUALITY LABOR Referents.*

*This Book is the testimony of how our VQL Referent, the laboratory IANTOSCA DENTAL SOLUTION, situated in AVELLINO, applies these procedures in its daily work.*

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*In my Dental lab we always say "we work hard to make you smile". Raffaele Iantosca, takes his degree in 1990 l'Istituto Giorgi di Avellino, and starts his path in dentistry at 12 years old, in a full denture specialized dental laboratory. Very fascinated in full denture has attended many courses with really famous italian and international prosthodontist such as: Odt. Bosshart, Odt. Zollo, Prof. Palla, Odt. Santanicola, Odt. Buongiovanni, Odt. Ugo Zambelli and many more... Since 1995 is the owner of Iantosca Dental Solutions his own lab, and had take lecture in many high school. Since 2016 has become Vertys Quality Labor and gives formation for Dental Technicians on Vertysystem techniques of Full Denture and Toronto Bridge.*

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Lab. Iantosca Dental Solution  
Avellino, Italy

# CASE

## *Complete Denture with Vertysystem*

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*The case described below, as common, is always a subject of a protocol developed by the company A.gree srl with the Vertysystem. In the removable denture, the coming of "furbo" flask and of its dedicated materials, has radically changed the way I work in laboratory, by optimizing the following steps of working:*

- cleaning and ease of use;*
- loyalty in the details reproduction;*
- resetting of ruptures and occlusal vertical dimension elevations;*
- simplicity in realising a denture clone.*

*Every definitive impression casting is realized on the chassis Split Cast. Once realized the mounting of the denture on the articulator, in order to achieve an optimized cosmetic, I use the dedicated waxes for marginal gingiva wax-up. Finally, the transformation in resin can take place using the pressing or injection technique.*

*Anyway the precision silicones will guarantee the reproduction of any detail. Only then can we be sure that we did a great denture.*

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*The Daily Work...*

## STEPS

A.01 Taking of the Upper and Lower impressions

A.02 Fabrication of the stone model into the Vertys Split Cast

A.03 Mounting of the stone model on the articulator Ar.go and wax up

A.04 Modeling of the marginal gingiva in wax

A.05 Application of the fluid silicone Vertys High Security

A.06 Registration with the precision silicone Vertys Orange Plus

A.07 Registration with the filling silicone Vertys Easy Putty

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EN *Taking of the Upper and Lower impressions.*

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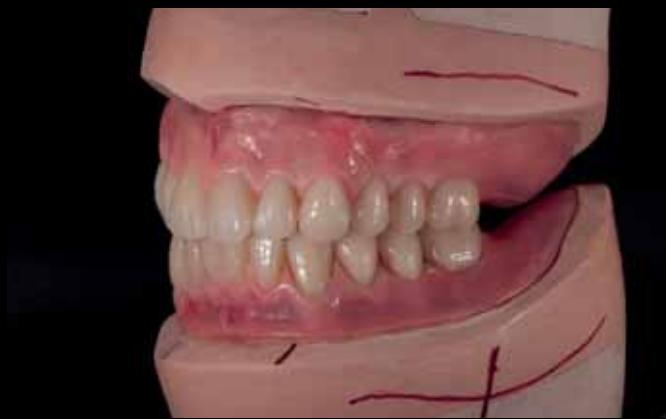
*Our occlusion rims, dentine colour, are made of easily modifiable material at moderate temperature and they are at the same time rigid, once cooled. The benefit of softening the rim in slightly warm water is that the wax becomes plastic but never pliable. The occlusion part will be clearer and well-defined. They have an adequate conformation to the functional placement over stabilised plate, picking up the thickness of natural teeth.*

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*A practical 80 g cone of the same colour of LASAGNA to complete in a practical and quick manner the prosthesis modelling. Also applying great quantities, at the cooling stage, this wax always keeps itself stable, without retraction. As with the LASAGNA utilisation, also with the TIMBALLO, the tooth remains clean and perfectly adherent to the plate.*

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*Two 40 g purple red and pale pink cones designed for the gingival characterisation. The two colours can be mixed together to create a personalised coloration.*

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*A wax bar of the same colour of LASAGNA and TIMBALLO to use with the wax warmer.*

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
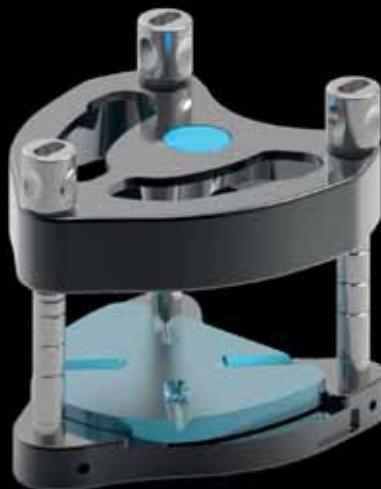
Vertys Extreme

## Used for:

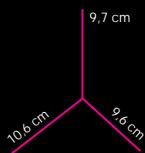- Complete Denture
- Temporaries
- Implant Bridge
- Partial Dentures
- Dentures Relines


## Dimensions:

## Used for:

- Class III Complete Denture
- Temporaries
- Implant Bridge
- Partial Dentures
- Dentures Relines
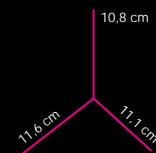
## Dimensions:

# Inclined Planes

Inclined plane 6°

Anodized aluminium inclined split  
base at 6°  
anodization: X-HART  
color: deep red

Inclined plane 9°

Anodized aluminium inclined split  
base at 9°  
anodization: X-HART  
color: deep red

Inclined plane 12°

Anodized aluminium inclined split  
base at 12°  
anodization: X-HART  
color: deep red

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**EN** Vertys Inclined Planes are inclined planes at 6° 9° 12° very useful to build the resin structure in Toronto Bridge. They give you the possibility to also save space bringing down the plaster model inside the Flask Furbo Alu Small / Big.

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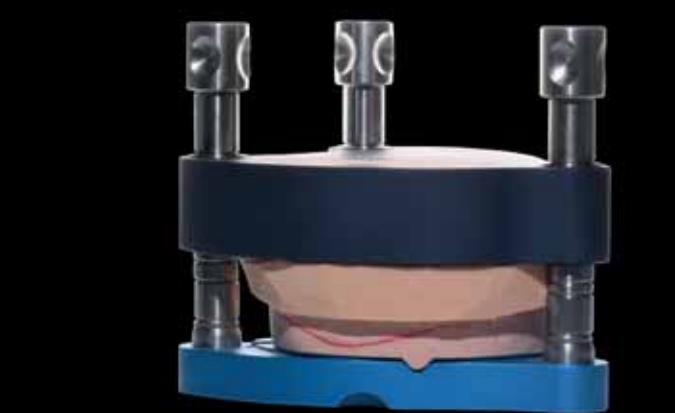
Smooth well the silicone Vertys Orang Plus surface during its application.

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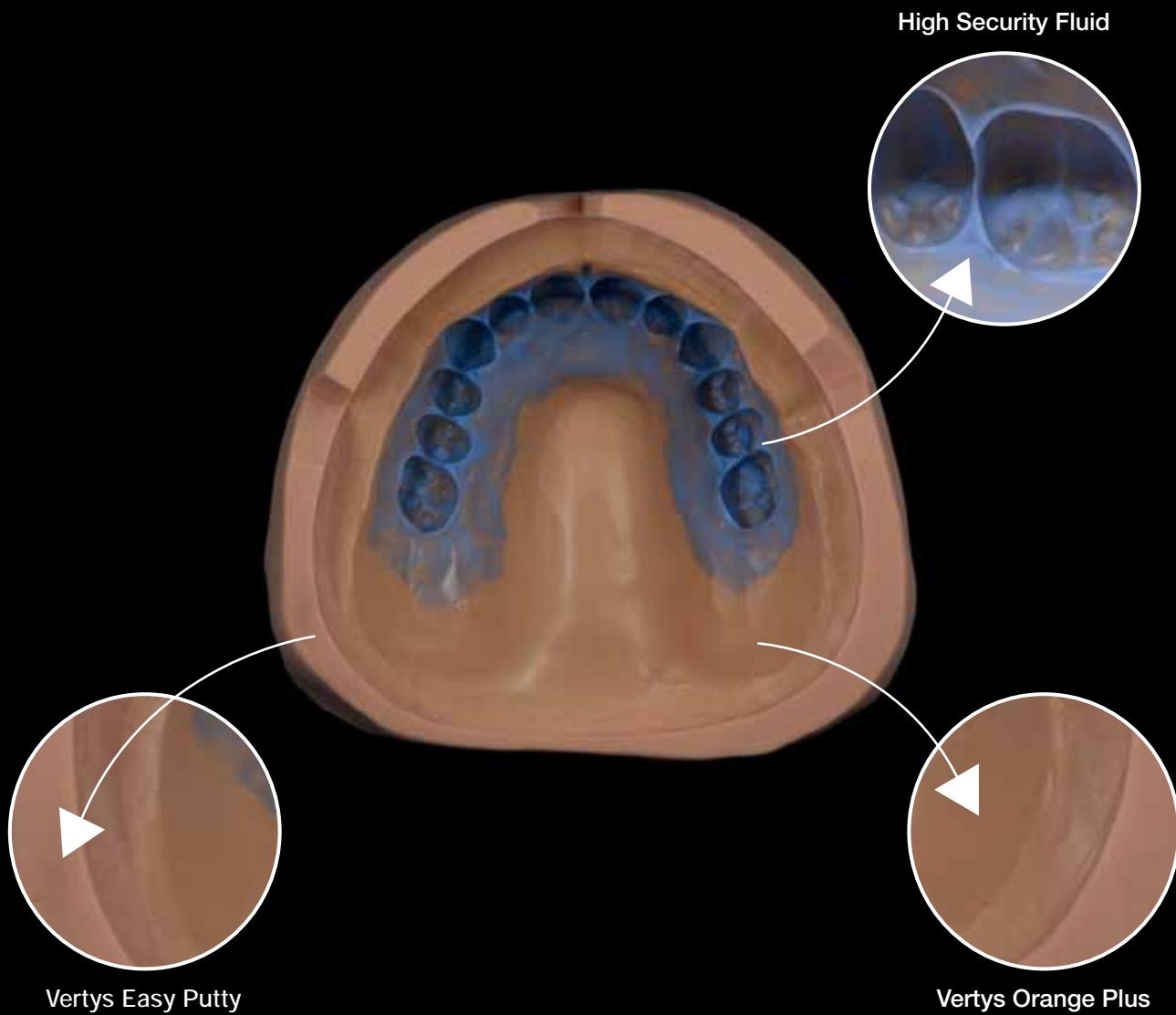
**EN** The registration silicones have to be perfectly integrated among them.

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**EN** High precision silicone, ideal for all types of registrations and, above all, for Implant Bridge, Full denture and ceramic Stamping. Having been designed not to absorb liquids, it offers excellent dimensional stability and chemically bonds with all VERTYSYSTEM® silicones.

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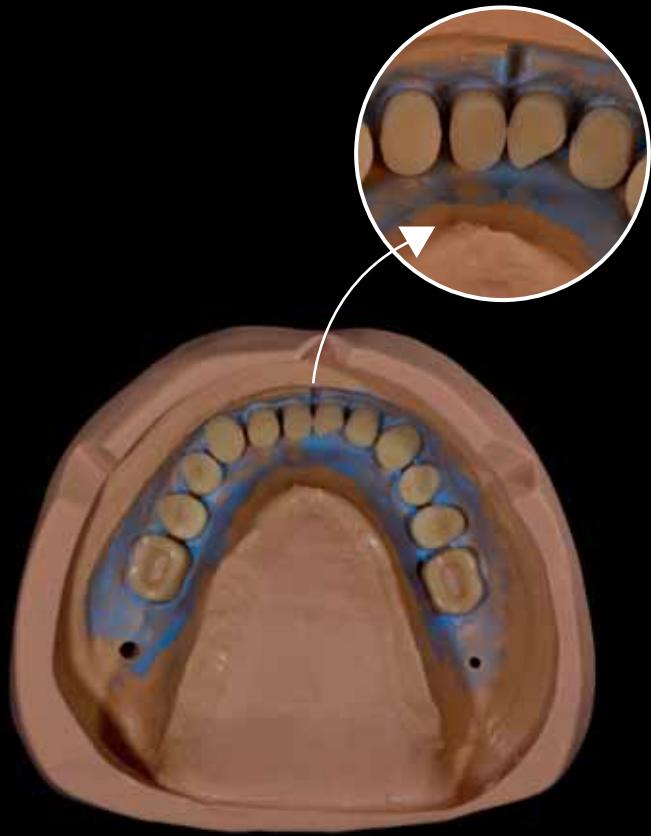
**EN** The "wildcard" of the system. VERTYS EASY PUTTY is a bonding silicone. It is often used in conjunction with VERTYS ORANGE PLUS for the production of temporary elements, complete prostheses, bites, surgical templates and ceramic Stamping, as well as for various other processes.

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# Acrylic resins with composite micro-pearls

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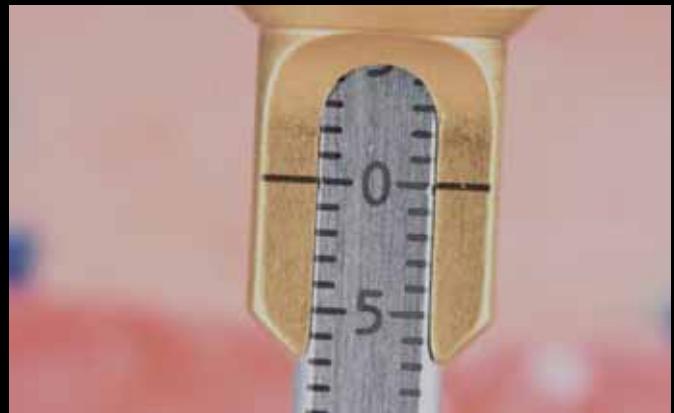
**EN** Enriched with a brand new molecule based on composite micro-pearsl VERTYS PROTHESIS HOT is a hot resin for removable prostheses which offers excellent quality and reliability. Ideal for use with "injection" and "pressure" techniques using the FURBO ALU BIG flask.

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**EN** This is an example of how to use the Vertysystem protocol.

This case was kindly grant by Dr. Rino Miraldi (Doctor in Naples) and Iantosca Dental Solution Lab (Avellino).

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**EN** Examples of refinished and polished complete dentures.

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**EN** *The Vertysystem has no limits, is up to you: which technique is the best press or injection, colors, using heat or cold curing resin... Whatever you decide to do, we have a protocol for it!*

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